

RC-Blue | RECIPROCATING SYSTEM

FEATURES:
RC-Blue - high efficiency and cutting performance achiever



Size	Apical Taper	Picture
R25	.08	
R40	.06	
R50	.05	

Material : Blue NiTi

Length : 21mm,25mm,31mm




Rotary speed : 350rpm Package : 3 pcs or 6 pcs per box/ blister, assorted/single size




RECIPROCATING MOVEMENT PATTERN


- Preparation the root canal with single file, convenient and quick
- Less working steps, more efficient and less time-consuming
- Suitable for curved and narrow root canal preparation
- New NiTi material enhances the flexibility and ability of anti-fracture
- Three sizes are available for different types of root canals

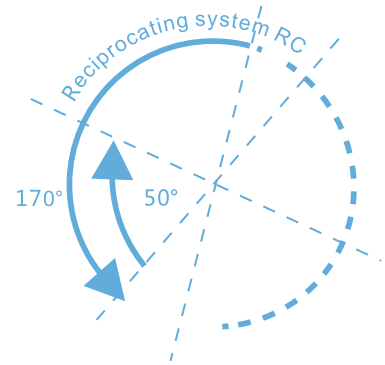

SPECIAL DESIGN IN APICAL

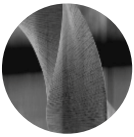

FAVORABLE HELICAL

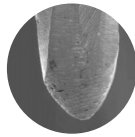

NON CUTTING TIP

- ⦿ 2 main cutting-edge designs of engine use nickel-titanium equipment.
- ⦿ With cutting effect apical - triangular cone, with cutting edge, Without cutting effect apical, which is designed in conical.
- ⦿ The unique apical design, which is non-consider cutting function but supervise and guide the device, reducing the risk of steps and fracture.

OPERATION GUIDE:

Taking photos of root canal tip before root canal preparation , so that it can display the canal diameters , lengths, root canal numbers and the bending of root canal.

FOR NARROW OR INVISIBLE CANAL

Opening the pulp, searching the root canal, using the 10# or #15 (stainless steel K file) to the working length.

The R25 RC-Blue File is most suitable in the great majority of cases.



R25 .08

FOR WIDE OR MEDIUM CANAL

#30 K file can reach the working length passively.

YES ✓




R50 .05

NO X →

#20 K file can reach the working length passively.

YES ✓

NO X

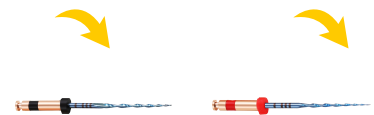


R40 .06

R25 .08

RC-BLUE OPERATING CASES:



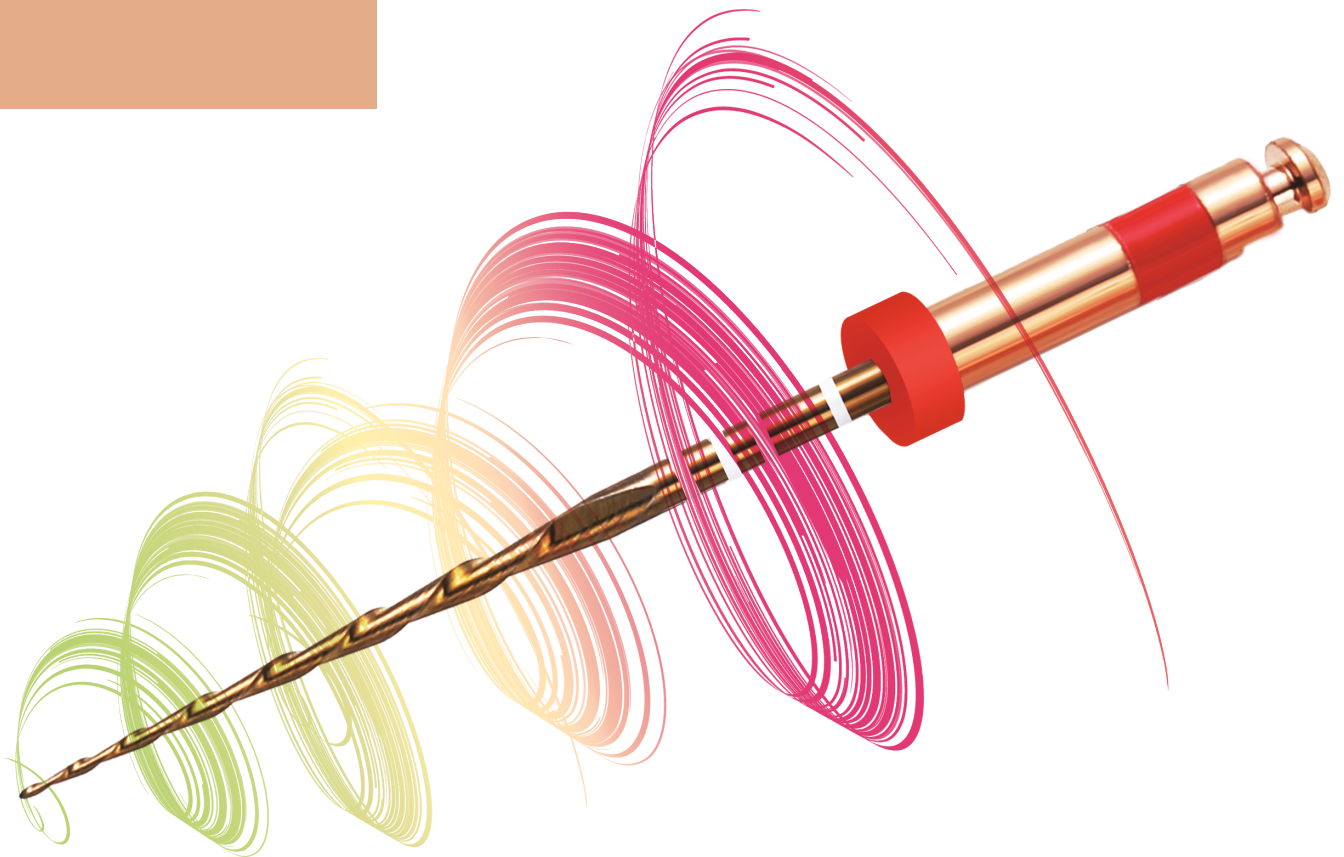

RC-Gold

RECIPROCATING SYSTEM

FEATURES:
RC-Gold - high efficiency and cutting performance achiever



Size	Apical Taper	Picture
R25	.08	
R40	.06	
R50	.05	

Material : Blue NiTi

Length : 21mm,25mm,31mm




Rotary speed : 350rpm Package : 3 pcs or 6 pcs per box/ blister, assorted/single size




RECIPROCATING MOVEMENT PATTERN


- Preparation the root canal with single file, convenient and quick
- Less working steps, more efficient and less time-consuming
- Suitable for curved and narrow root canal preparation
- New NiTi material enhances the flexibility and ability of anti-fracture
- Three sizes are available for different types of root canals

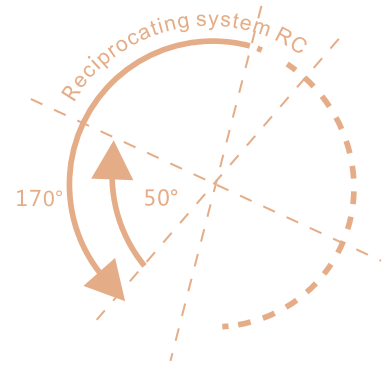

SPECIAL DESIGN IN APICAL

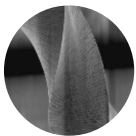

FAVORABLE HELICAL

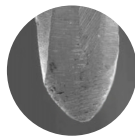

NON CUTTING TIP

- 2 main cutting-edge designs of engine use nickel-titanium equipment.
- With cutting effect apical - triangular cone, with cutting edge, Without cutting effect apical, which is designed in conical.
- The unique apical design, which is non-consider cutting function but supervise and guide the device, reducing the risk of steps and fracture.


OPERATION GUIDE:

Taking photos of root canal tip before root canal preparation, so that it can display the canal diameters, lengths, root canal numbers and the bending of root canal.

FOR NARROW OR INVISIBLE CANAL

Opening the pulp, searching the root canal, using the 10# or #15 (stainless steel K file) to the working length.

The R25 RC-Gold File is most suitable in the great majority of cases.

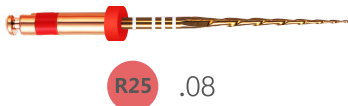

FOR WIDE OR MEDIUM CANAL

#30 K file can reach the working length passively.

YES ✓




NO X →

#20 K file can reach the working length passively.

YES ✓

NO X

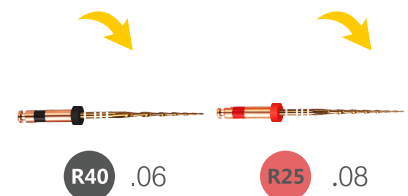

RC-GOLD OPERATING CASES:

