



# T-Pro | BEYOND PERFECTION

Material : Gold NiTi    Length : 21mm,25mm,31mm    Rotary speed : 200-350rpm    Package : 6pcs/blister



Cross Section :



### Protocol for use

- Explore the canal using #8/10 or #15 K file, always irrigate the canal when needed.
- Widen the orifice of the canal by ST file.
- Glide path preparation with PG file for curved canal if necessary.
- Remove and shape with system files E1 & E2 to the working length.
- For severely curved canals, size E3 & E4 are required.

### Advantages

- New generation files combined Heat-treatment files with heat-activated.
- 200% improved in the durability, cutting efficiency and fracture resistance.
- Files ST/PG/E1/E2 are processed with Heat-treatment while E3/E4 with Pre-bendability.
- Superior quality and best cost-effective for you!



The logo features a stylized white 'D' icon on the left, followed by the word 'perfect' in a lowercase, italicized sans-serif font, and a registered trademark symbol (®) to the right.

**D perfect®**

*Perfect Endo.*

[www.dental-perfect.com](http://www.dental-perfect.com)