



Material: NiTi
 Standard : Silver, Gold
 Length: 21, 25, 31 mm
 16 mm working part (21 or 25 mm length)
 21 mm working part (25 or 31 mm length)
 Rotary speed: 250-360 rpm
 Torque: 2.0~3.0 N/cm
 Package: 4 or 6 pcs per box/blister assorted/single size

Basic Sequence

● 10/.04 ● 15/.05 ● 20/.06 ● 25/.06

Shaping of Larger Canals

● 30/.05 ● 35/.04 ● 40/.04 ● 45/.04 ● 50/.04 ● 60/.04

Shaping of Larger Canals

● 25/.07 ● 30/.06 ● 35/.06 ● 40/.06



MT BLUE

Features

Compatible to traditional MTwo, PERFECT MT Blue Files with a cross section of S Shape which progressively increasing pitch avoids screw-in effect. Safety non-cutting tip minimizes risk of ledge formation. Better cutting efficiency, cleverly flexibly for different types canals shape.

Protocol for use

MT Blue File is one sequence for all types of root canal. Explore the canal by ISO #8/#10 hand instrument (K/C File), always irrigate the canal when needed. Widen the orifice of the canal by Starter or Burs. Glide path preparation with #10 & #15 MT-Blue Files. Remove and shape with system files #20 & #25 MT-Blue Files to the working length. For severely bigger canals, size #30, #35, #40 are required.

